

# CM ProSlide™

CM ProLine

Slide Board



- ✓ EASY TO USE
- ✓ HYGIENIC
- ✓ ERGONOMIC
- ✓ DURABLE



Stretch the clothing and the bed sheet. Hold the sheet with one hand and grab the ProSlide with the other hand according to the picture.

Slide in the ProSlide halfway underneath the patient, between the sheet and the clothes or between the sheets if there are two of them. The lustreless side of ProSlide slides best against vinyl or other plastic materials. Grab the sheet (not the ProSlide) and pull the patient over to the table/trolley or into the bed.

NOTE: - Never slide in ProSlide directly against the patient's skin.  
- Be sure that trolley- or bedwheels are locked before sliding the patient.

#### PRODUCT INFORMATION:

ProSlide is made of 3 mm solid, flexible HD-polyeten with low friction.

ProSlide is used for transferring of patients between x-ray/OR-tables and trolleys/beds.

It can also be used as a lifting tool in connection with bedside x-ray. Prehospital tool for transferring of trauma- and emergency patients and for application for T-Pod.

Gives no artefacts on x-ray.

Cleaning with soap-alkohol solution.

Can be made in special size/shape.

Size: 86x39 cm Weight: 0,78 kg

Article nr: CMPRO



#### MANUFACTURER:

Comfort Medical AB  
Hjalmar Brantings väg 11G  
SE - 261 43 Landskrona  
SWEDEN  
Tel: 0733 109020  
0706 815141  
Mail: info@comfortmedical.se  
www.comfortmedical.se

COMFORT medical