

CM ProSlide

CM ProLine



PRODUCT INFORMATION:

ProSlide is made of 3 mm solid, flexible HD-polyethylene with low friction. ProSlide is used for transferring of patients between X-ray/OR tables and trolleys/beds.

Stretch the patients clothing or the bed sheet. Hold the sheet with one hand and grab the ProSlide with the other hand according to the picture above. Slide two ProSlides halfway under the patient. Grab the sheet (not the ProSlide) and pull the patient over to the table/trolley or into the bed.

- **EASY TO USE**
- **ERGONOMIC**
- **HYGIENIC**
- **DURABLE**

Article no: CMPRO

Size: 86x39 cm

Weight: 0,8kg

Material: 3mm flexible HD polyethylene

Cleaning and disinfection:

Clean CM ProSlide with alcohol and/or soap solution



2022-04-01 (Edition:2)

Distributor: **COMFORT** medical

Manufacturer: Bima Plastteknik AB, Trålvägen 5,
43274, Träslövsläge, Sweden

Comfort Medical AB | Hjalmar Brantings väg 11G | 26143
Landskrona | Sweden | +46 (0)733-109020 |
Info@comfortmedical.se | www.comfortmedical.se