

CM T-Fix

CM ProLine



Easy and quick fixation of the head.



Each set is delivered complete with hood, cushion and two cheek-forehead straps.

PRODUCT INFORMATION

CM T-Fix is a single use head fixation device for trauma mattresses. Each package contain a hood, two cheek-forehead straps and a fixation cushion.

CM T-Fix offer an easy fixation of the head by the fixation cushion. The cushion is applied by velcro directly onto the hood/mattress.

- HEAD FIXATION
- HYGIENIC
- TIME SAVING

Name: CM T-Fix head fixation

Article no: CMTFIX

Packing size: 25pc/box

Cushion size: 27x27x7 cm

Recycling: No disassembly required. Sort as combustible waste. Contaminated waste is sorted in accordance with the hospital's guidelines.

CE  single use only

2023-04-11 (Edition:2)

Distributor: **COMFORT** medical

Manufactured by: Bima Plastteknik AB, Trålvägen 5,
43274, Träslövsläge, Sweden

Comfort Medical AB | Hjalmar Brantings väg 11G | 26143
Landskrona | Sweden | +46 (0)733-109020 |
Info@comfortmedical.se | www.comfortmedical.se

User Guide CM T-Fix



1. Pull the hood over the top end (head) of the trauma mattress.



2. Tighten the velcro straps firmly around the top end.



3. Place the cushion tight around the patient's head. The velcro fixates on the hood. Long hair should be pulled under the neck collar if possible.



4. Fixate the patient's head with the two white cheek-forehead straps. The straps should be fixated against the velcro on the hood and the cushion.