

CM TraumaCover

CM ProLine



PRODUCT DESCRIPTION

CM Trauma Cover is a single use cover for trauma mattresses. The package contain cover, two cheek-forehead straps and a fixation cushion for the head. The material properties minimize heat loss for the patient and contributes to better hygiene standards. The inside of the cover is absorbing, the outside is waterproof. Therefore the use of CM Trauma Cover drastically reduce the need of cleaning.

The head fixation allow a simple and secure fixation of head and neck. The head fixation is integrated with the Cover. The cushion is easily applied onto the cover.

CM Trauma Cover is available in child version and adjusted to the CM Trauma Mattress Mini.

- HEAD FIXATION
- HYGIENIC
- TIME SAVING
- WARM

Adult cover for CM Trauma Mattress:

- **Size:** 225X162 cm
- **Article no:** CMCOVER
- **Packing size:** 15 pc/box

Child cover for CM TraumaMattress Mini:

- **Size:** 140x120 cm
- **Article no:** CMCOVERM
- **Packing size:** 15 pc/box

Recycling: No disassembly required. Sort as combustible waste. Contaminated waste is sorted in accordance with the hospital's guidelines.

CE (2) single use only

2023-04-11 (edition:2)

Distributor: **COMFORT** medical

Manufacturer: Bima Plastteknik AB,
Trålvägen 5, 43274, Träslövsläge, Sweden

Comfort Medical AB | Hjalmar Brantings väg 11G | 26143
Landskrona | Sweden | +46 (0)733-109020 |
Info@comfortmedical.se | www.comfortmedical.se

User Guide CM TraumaCover



1. Pull the cover over the top (head) end of the trauma mattress. Stretch from the foot end.



2. Tighten the velcro straps firmly around the top (head) end.



3. Place the cushion tight around the patient's head, the velcro fixates on the cover. Long hair should be pulled in under the neck collar if possible.



4. Fixate the patient's head with the white cheek-forehead straps. The straps should be fixated against the velcro on the cushion and cover.

